

TECHNICAL DATASHEET

PRODUCT NAME

DISPO ICE SPRAY

PRODUCT CODE

SP400DISPONN - SP200DISPONN (NN: number of cans in the box)

PRODUCT DESCRIPTION

Ice spray. Cans content: 200 mL and 400 mL



TECHNICAL AND MATERIAL FEATURES

Ingredients: Isobutane, propane, butane, methylal, isopropyl myristate, menthol

LATEXFREE

The lot is indicated on the top of the can.

Expiration date: 5 years from the date of production. It is shown on the bottle cup.

CLASSIFICATION

Medical Device Class IIA

Classification according to Annex VIII, rule 9 of Regulation (EU) 745/2017 and subsequent amendments

MANUFACTURER

Dispotech srl Via Al Piano 29 23020 Gordona (SO)

Tel. 0343.36711 Fax 0343.36567 e-mail: info@dispotech.it

INDICATIONS FOR USE/INSTRUCTION FOR USE

Used for relieving pain caused by injuries, scrapes, falls or sprains, and in cases requiring cryotherapy and application of cold. Thanks to its anaesthetic effect induced by cold on the injured area, sports activities can be resumed quickly.

Shake before use. Spray product on the affected area from a distance of at least 20 cm for 3-4 seconds. Repeat application if necessary. The mechanical anaesthesia induced by cold eliminates pain temporarily.

STORAGE

Store at room temperature, away from heat, open flames or sparks.

Keep away from sunlight and do not expose to temperatures exceeding 50°C/122°F

PCS/BOX 12 or 24 LAYERS PALLET

CARTON SIZE PALLET SIZE

CARTON LAYER PALLET WEIGHT

CARTON PALLET

SYMBOLOGY

Symbology according to the norm UNI CEI EN ISO 15223-1:2021

REVISION DATE APPROVED BY 29/07/2024 - REV. 07 DG



Cap. Soc. € 1.500.000,00 i.v. - P. IVA 00672170149 - R.E.A. 47213 C.C.I.A.A. di SO - Uff Reg. Imp. SO 00672170149



MD

